# DUROLANE<sup>®</sup>

## INSTRUCTIONS FOR USE

Contents

Each ml contains:

Hyaluronic acid, stabilized 20 mg  
Phys. sodium chloride solution, pH 7 q.s.

Description

DUROLANE<sup>®</sup> is intended to be used for intra-articular injection for the symptomatic treatment of mild to moderate knee or hip osteoarthritis. Additionally, DUROLANE<sup>®</sup> is intended to be used for intra-articular injection for symptomatic treatment of mild to moderate osteoarthritis of the shoulder, elbow and wrist following arthroscopic procedures. It should be injected by an authorized physician, or in accordance with local legislation.

DUROLANE<sup>®</sup> contains 20 mg/ml stabilized non-animal hyaluronic acid in buffered physiological sodium chloride solution pH 7. DUROLANE<sup>®</sup> is a sterile, transparent viscoelastic gel supplied in a 3 ml glass syringe. The product is single use only.

Hyaluronic acid is identical in all living organisms. It is a natural polysaccharide that is present throughout the tissue of the body, with particularly high concentrations in the synovial fluid and the skin. DUROLANE<sup>®</sup> is composed of a mixture of hyaluronic acid produced from pigs following arthroscopic procedures. It should be injected by an authorized physician, or in accordance with local legislation.

DUROLANE<sup>®</sup> contains 20 mg/ml stabilized non-animal hyaluronic acid in buffered physiological sodium chloride solution pH 7. DUROLANE<sup>®</sup> is a sterile, transparent viscoelastic gel supplied in a 3 ml glass syringe. The product is single use only.

**Mode of Action**

The acidic hyaluronic acid constitutes a natural part of the synovial fluid and acts in the joints both as a lubricant of cartilage and ligaments and as a shock absorber. Injections of hyaluronic acid in the joint to restore the viscosity and elasticity can diminish the pain and improve the mobility of the joint.

**Dosage**

DUROLANE<sup>®</sup> is a single injection, single dose preparation and should only be injected once into the knee, hip or shoulder joint. The recommended dose is 3 ml for knee, 1-2ml for intermediate joints (e.g., elbow, ankle) and approximately 1ml for small synovial joints (e.g. thumb, temporomandibular joint, face).

**Indications**

Symptomatic treatment of mild to moderate knee or hip osteoarthritis. In addition, DUROLANE<sup>®</sup> has been approved for the symptomatic treatment associated with mild to moderate osteoarthritis of the shoulder, elbow and wrist, fingers, toes, and the temporomandibular and facet joints. DUROLANE<sup>®</sup> is also indicated for pain following joint arthroscopy either in the presence of osteoarthritis or subsequent to general surgical repair within 3 months of the procedure.

**Contraindications**

None known.

**Warnings**

DUROLANE<sup>®</sup> should not be injected if the synovial joint is infected or severely inflamed.

DUROLANE<sup>®</sup> should not be injected if there is an active skin disease or infection present at or near the injection site.

DUROLANE<sup>®</sup> should not be injected intravascularly or extra-articularly or in the synovial tissues or capsule.

Do not reconstitute DUROLANE<sup>®</sup> as this may damage the product.

**Precuations**

DUROLANE<sup>®</sup> should be used with caution in patients with venous or lymphatic stasis present in the leg.

DUROLANE<sup>®</sup> has not been tested in pregnant or lactating women or in children under 16 years of age.

A separate syringe of DUROLANE<sup>®</sup> must be used for each patient's own injection.

Do not reconstitute DUROLANE<sup>®</sup> as this may damage the product.

**Precautions**

DUROLANE<sup>®</sup> should not be injected with prudence since the patients presenting a same veinous or lymphatic disease in the jamb.

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